Rollator / Rollator / Rollator / Andador / Deambulatore

RLT01

ORIGINAL AUFBAUANLEITUNG / MODE D'EMPLOI ORIGINAL / ORIGINAL INSTRUCTION MANUAL /
MANUALE DI ISTRUZIONI ORIGINALE / MANUAL DE USUARIO ORIGINAL

LIEBE KUNDIN, LIEBER KUNDE,

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BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

- Your rollator has been designed to assist you when walking and carrying small items of shopping using the bag provided. The bag was designed specifically to fit your rollator, please only use the bag supplied for your rollator. Do not use or combine bags and/ or baskets from other manufacturers.
- When not in use, you can simply fold the rollator and store away until you require it next. The range of rollators are designed to be used both inside and outside the home.

ASSEMBLY INSTRUCTIONS

- Carefully remove all packaging, including the cardboard tubes covering the knurled tube part of the brake assembly. Check the product thoroughly for damage or obvious faults.

If you suspect the unit to faulty, please do not use, but contact your supplier immediately for service.

COMPONENTS OF THE ROLLATOR

ASSEMBLY PROCEDURE

Figure 1

Step 1:

Press the red button to detach the front wheel, the red button is located on top of the wheel. Repeat on the other wheel.

To assemble the front wheel, push the front wheel into the receptacle place, the front wheel is fully fastened when you hear the click sound.(Figure 1)

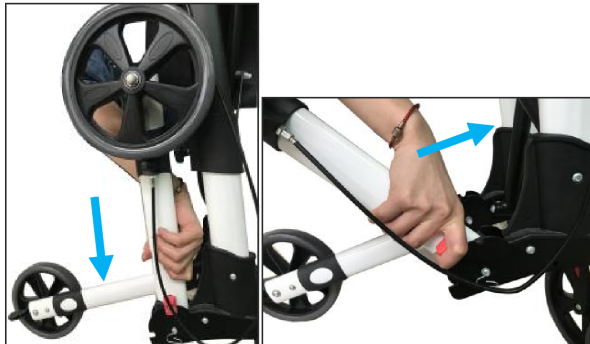
Figure 2

Step 2:

Unfolding rear wheel: Press the button on both sides and pull up, then move the rear wheel pole downward until reset on the next button position. Repeat for the second rear wheel.

Folding rear wheel: Press the button on both sides and pull up, then move the rear wheel pole upward until reset on the next button position. Repeat for the second rear wheel.

Figure 3

Step 3:

Attach the handles by pressing the locking pin on the base of the handle assembly and sliding into the receptacle on the main frame. Repeat on the other side. (Figure 3)

Figure 4

Step 4:

Attach the bag to the hook of the seat frame. The shopping bag can be detached easily when using. (Figure 4)

Figure 5

Step 5:

Unfold the rollator by pressing down on both sides of the seat frame. (Figure 5)

Figure 6

Step 6:

To fold, pull seat upwards by the black handle on the seat plate as show in (Figure 6) . The rollator can now be folded completely.

OPERATING THE BRAKES

Figure 7

1. To stop, gently apply a gripping pressure to both brakes simultaneously (figure 7), this will slow down and stop your Rollator.

Figure 8

2. The brakes on your Rollator have a parking option (figure 8). To operate the parking option, push downwards on both brakes simultaneously. You will feel

the brake make a very slight click. Your brakes are now in the 'lock' position.

3. To unlock, simply pull up the brake again, you will hear a slight click as the 'park' disengages.

BRAKE ADJUSTMENT

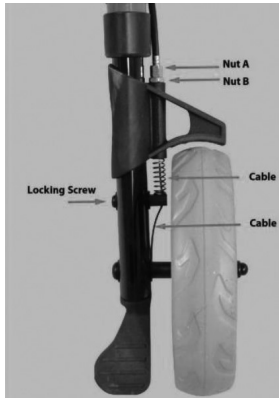
Figure 9

Your Rollator brake system will require periodic adjustment. To adjust the brakes loosen Nut B and then screw/unscrew Nut A. When adjusted re-tighten Nut B. If further adjustment is required then loosen the Locking Screw and then using a pair of pliers pull the cable down by approximately a millimetre. Re-tighten Locking Screw. The brakes can then be adjusted further by loosening Nut B and screwing/unscrewing Nut A as before (Figure 9)

CARE AND MAINTENANCE & YOUR OBLIGATION

1. Only use your Rollator for the purpose for which it intended, which is as an aid to walking. It is intended your rollator is only used on standard pavements or at home. It should not be used on rough terrain or pot-holed surfaces. The use of your rollator on these surfaces can seriously compromise your safety and may damage your rollator beyond repair.
2. Keep your Rollator clean by wiping down with a damp cloth with a mild detergent. Do not under any circumstances use an abrasive or solvent based cleaner, as this will seriously damage the rollator and void your warranty.
3. Wipe off all surfaces with a dry cloth and use a water dispersing agent such as WD40 on all metal joints.
4. Check that all wheels and the folding mechanism function correctly and that all wheels move freely. Pay particular attention to the front wheels, ensuring that they rotate freely and without obstruction and that they are attached securely to the frame.

Withdraw your Rollator immediately if any of the wheels become difficult to turn or do not feel securely attached to the frame or if you suspect damage or any other fault.

5. Monthly Checks - check that all locking pins are properly secure .
Check that no components are loose. Check the adjustment to your brakes .
Check castors.
6. Do not fit any parts or accessories to your rollator manufactured by another company. To do so will annul your warranty and may seriously compromise your safety.
7. We suggest that you check your rollator visually every day. However we recommend that you should have your rollator serviced every three months or less if used by, or issued to, various users.

CLEANING

Clean your Rollator using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any Concerns.

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Hergestellt für: FF Europe E-Commerce GmbH, Dr.-Robert-Murjahn-Str. 7, 64372 Ober-Ramstadt / Deutschland
Fabriqué pour: FF Europe E-Commerce GmbH, Dr.-Robert-Murjahn-Str. 7, 64372 Ober-Ramstadt / Allemagne
Produced for: FF Europe E-Commerce GmbH, Dr.-Robert-Murjahn-Str. 7, 64372 Ober-Ramstadt / Germany
Fabricado para : FF Europe E-Commerce GmbH, Dr.-Robert-Murjahn-Str. 7, 64372 Ober-Ramstadt / Alemania
Prodotto per: FF Europe E-Commerce GmbH, Dr.-Robert-Murjahn-Str. 7, 64372 Ober-Ramstadt / Germania